

Trilogy202




Quick start guide

- 1 Overview
- 2 Circuit setup
- 3 Setup screen
- 4 Prescription setup

1

Overview

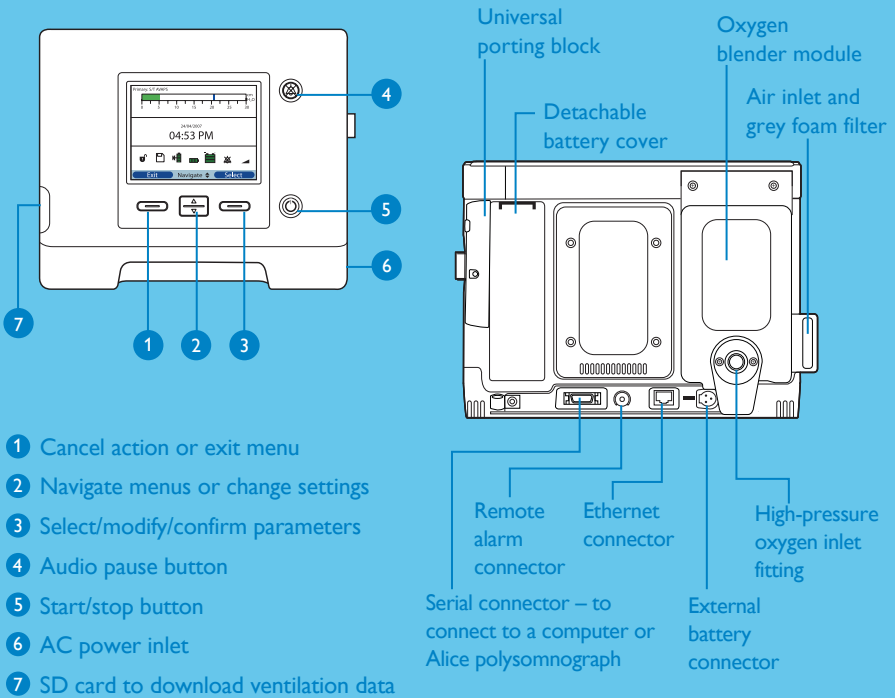

Symbols and icons



Full access mode



SD card inserted in the ventilator



Audio pause active



Ramp active



Status of optional detachable battery



Status of internal battery



Status of external battery




Battery is charging

2

Circuit setup


Before operating the Trilogy 202, you need to configure the circuit type.

Passive exhalation port circuit

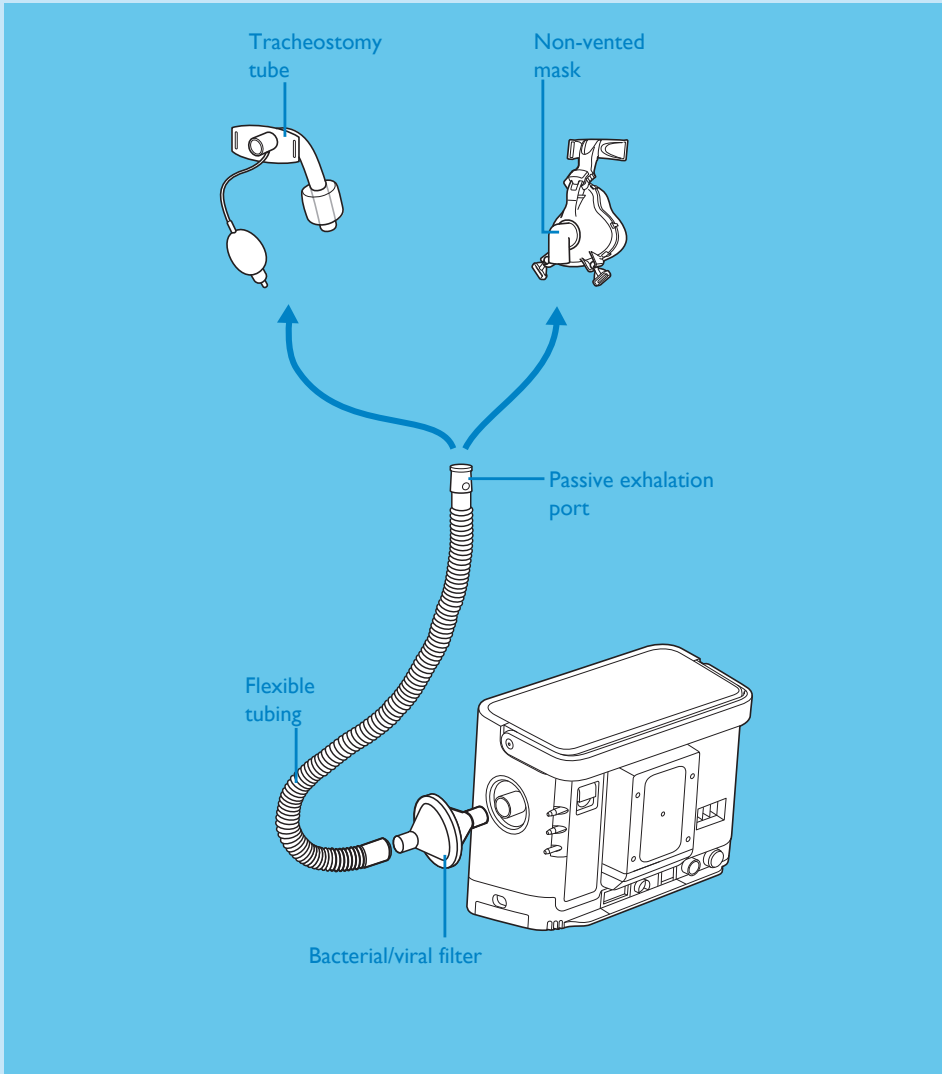


From the Setup screen select "Passive" for circuit type.

Active exhalation valve with proximal airway pressure circuit

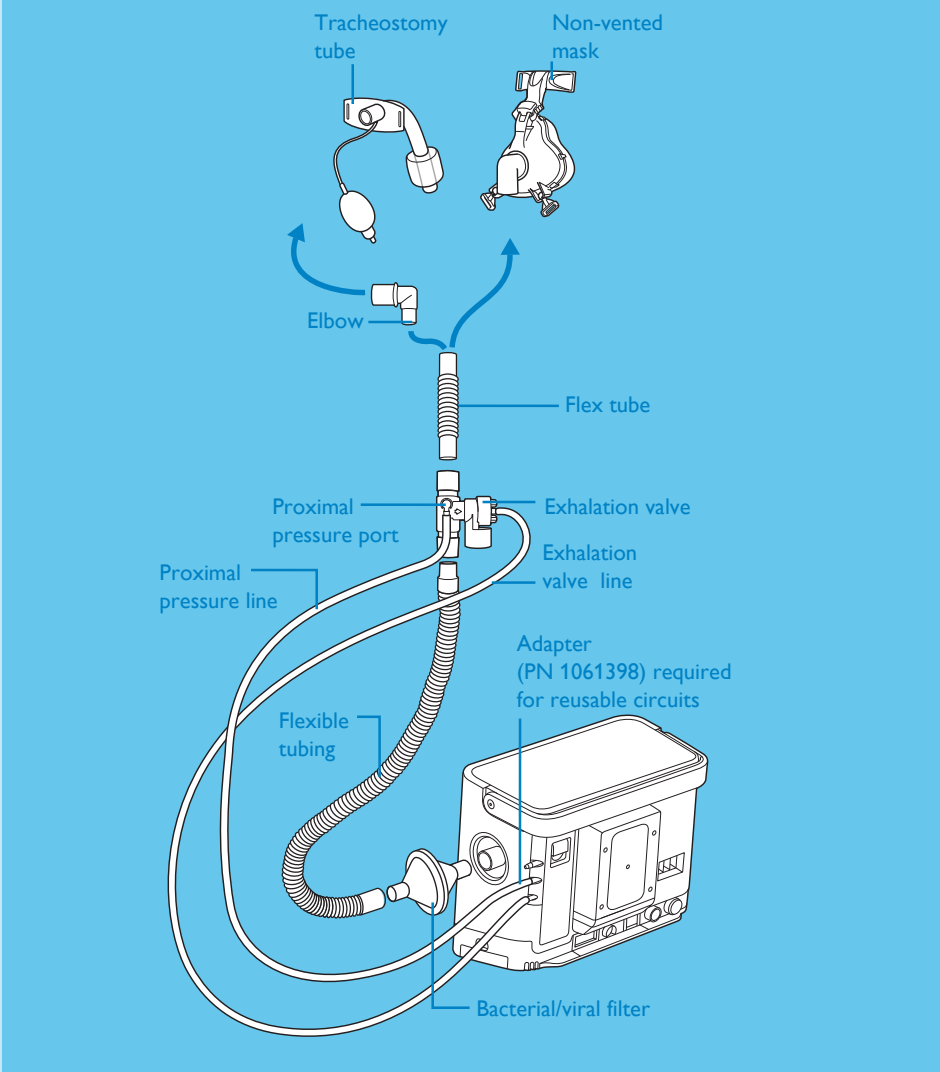

From the Setup screen select "Active PAP" for circuit type.

Active exhalation valve flow sensor circuit

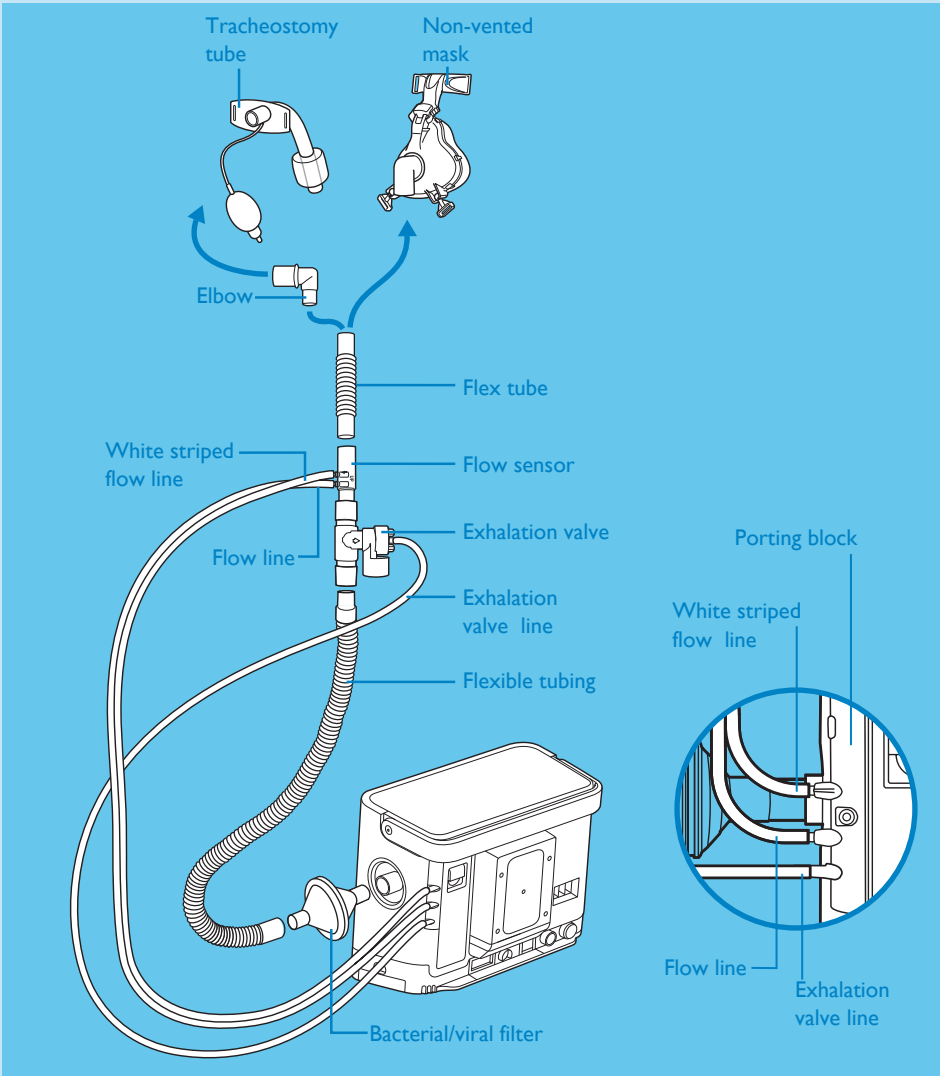


From the Setup screen select "Active flow" for circuit type.



3


Setup screen

Configuring the circuit type (active or passive)

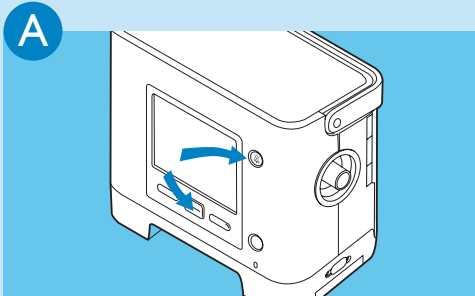
Make sure the Trilogy 202 is turned off before taking the following steps




Press and hold  and  for a few seconds to access the **Setup** screen

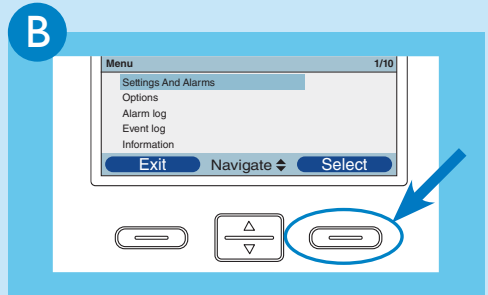

Select **Settings And Alarms**

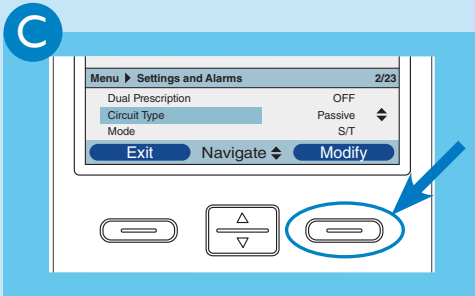


Navigate to **Circuit Type** and press 

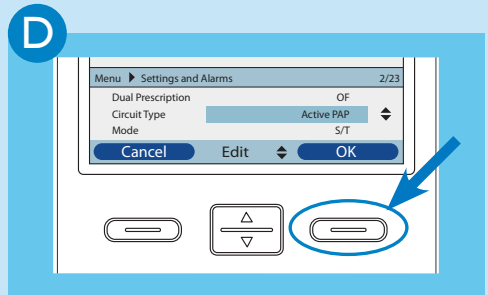


Use  to choose **Active PAP**, **Active flow**, or **Passive** and then press  twice to confirm

Setting menu access (May be performed while on or off the patient)


To enable full access to the settings

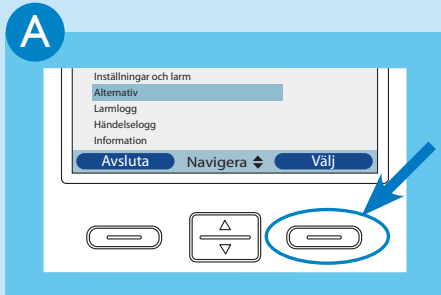


Navigate to **Options** and press **Select**

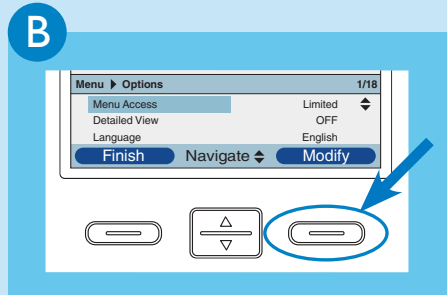


Press **Modify** to enter **Menu Access**

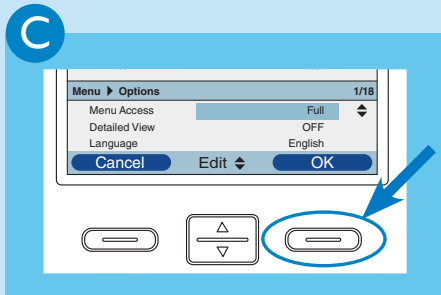

Use **▲ ▼** to choose **Full** or **Limited Access** and press **OK** to confirm

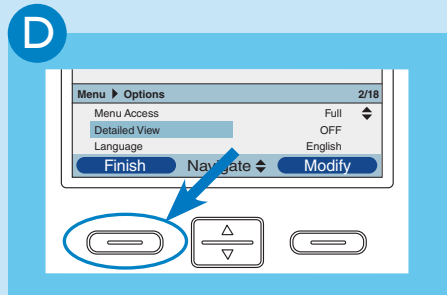

Press **Finish** and **Exit** to return to **Initial Screen**

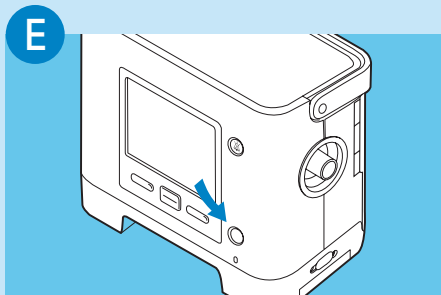


Press **⏻** to begin therapy **or** continue navigating through the menu to set up the ventilation parameters

4


Prescription setup

A

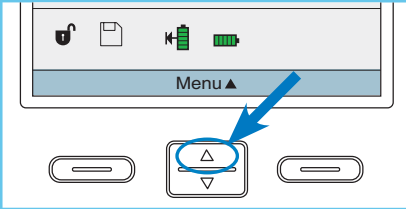


From main screen, press **Menu**


B

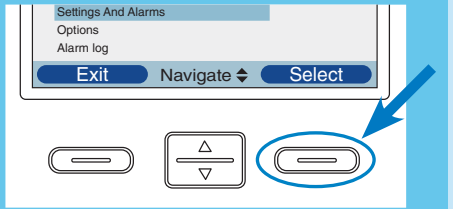


Select **Settings And Alarms**

E

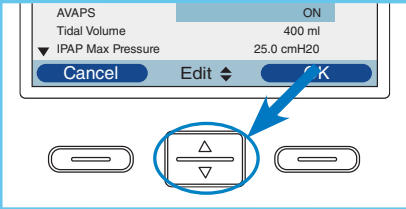


Use  to change the value or status of setting

F

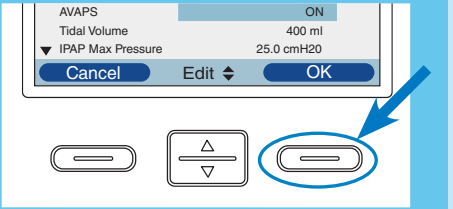


Press **OK** to validate new setting


Dual Prescription

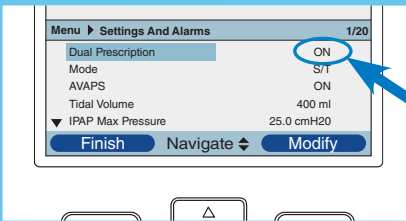



If **Dual Prescription** is enabled, both prescriptions need to be adjusted as previously explained

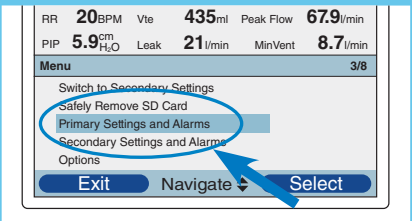


Set both primary and secondary prescriptions

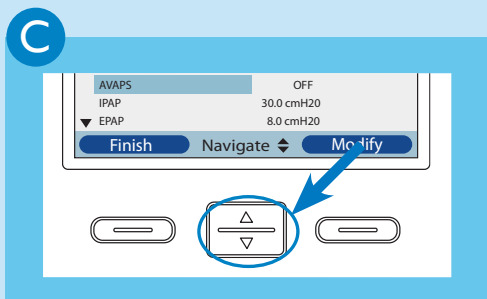

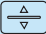Scroll up and down using  to navigate through the settings and alarms

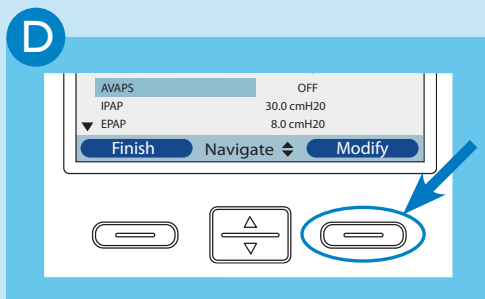

Press **Modify** to change the parameter highlighted in blue

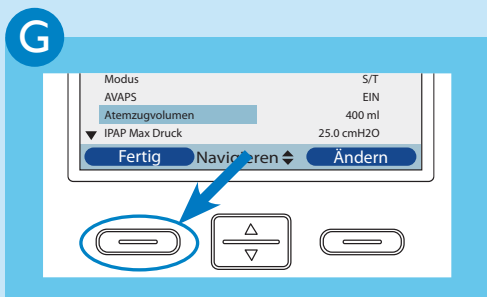

Press **Finish** and **Exit** to return to the main screen

Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA



Respironics Deutschland
Gewerbestr. 17
82211 Herrsching, Germany



1068642
JH 1/28/10